

# Reusable Face Mask

Antiviral-bacterial Refill Filters

**eGee-Pro®**  
A division of DiGIPAS group



- Max. protection against **GERMS** & **DUST** (3 Ply antiviral-bacterial filter)
- Hypoallergenic (BPA Free)
- Good Fit Design (Ergonomic face contour)

## Compliance Standards :

- Europe Surgical Mask. : EN14683:2019
- Europe Respiratory Filter : EN149:2001-A1:2009
- USA Plastic Material : BPA (EC) No. 1895/2005



FDA approved plastic material  
BPA (EC) No. 1895/2005

## FEATURES



Washable & Reusable



Separation between  
nose & mouth  
chamber design




Comfort & Safe to skin




3ply Refill anti  
bacterial Filters

Patented Face Mask Technologies

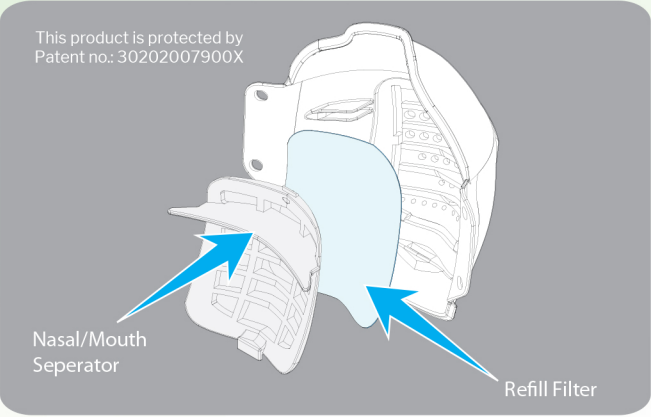

- REFILL Germ Filter technology
- Separating Nasal & Mouth Compartments Design

Bacterial Filters

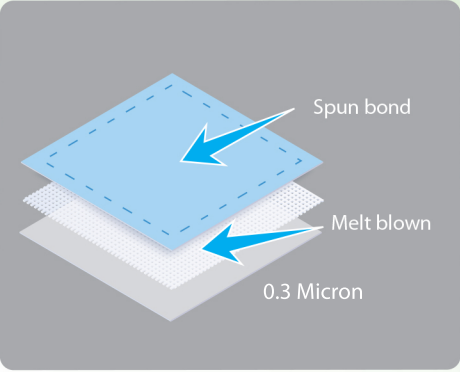

50 Pcs  
Refill Catridge

- 3 Ply fine layers of Melt blown & Spun bond Non-woven antibacterial filtering materials


Easy steps to sanitize eGee-Pro Reusable Mask

Using soap/detergent sanitizing



Wash your mask in clean warm (around 30°C) soapy water, making sure you rinse all components thoroughly before allowing them to dry naturally.

Hot water sanitizing

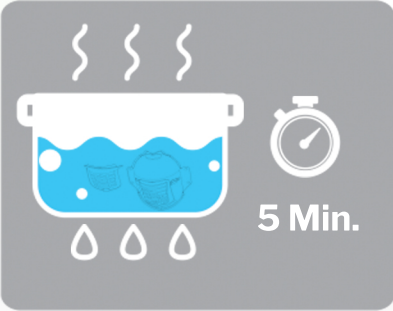

1. Heat the water to boil
2. Put the plastic reusable face mask in water for 5 - 10 minutes to sanitize

**CAUTION:**  
Filter cartridges are NOT for reuse, remove the used filter and replace it with a new filter.  
  
Not suitable for children use.

Quality Compliance Standards

- Manufactured in Cleanroom facility certified to ISO Class 8 (100.000)
- Tested to comply with EN14683:2019 and ASTM F2100 Bacterial Filtration Efficiency (BFE)
- Tested to comply with Respiratory Filtering Standard EN149:2001-A1:2009
- Proucts are UV sterilize and packed

Product Properties & Performance

Europe: EN14683 and USA: ASTM F2100 Standards	Bacterial Filtration Efficiency (BFE) @ 3.0 micron	≥ 99%
Europe Respiratory: EN149:2001-A1:2009 Standard	Particale Filtration Efficiency (PFE) @ 0.3 micron	≥ 99%

Product of Singapore

**JSBTECH PTE LTD**  
100 Pasir Panjang road, 100 Pasir Panjang  
#06-02 Singapore 118518

Dimension (mm)

