
## Mask overview

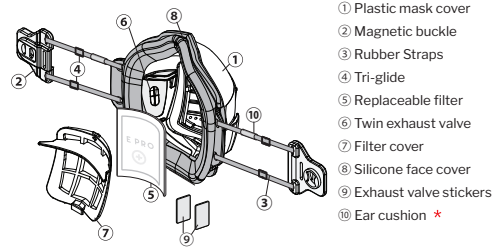
## Note:


\* Stickers are for sealing off exhaust valve to prevent the wearer from contaminating others.

## Adjusting Straps

To adjust length of the strap, pull the strap & move the ④ tri-glide to the left or right to suit user preference.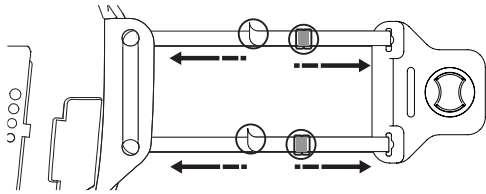

## How to change filter

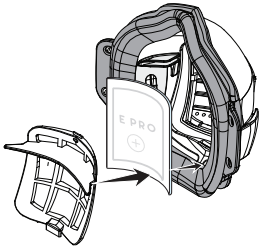- 1 Pull out filter compartment cap & insert filter cartridge with blue side facing outward from user's face.
- 2 Close back the filter compartment cap by inserting the bottom first and push forwards until you hear a click.

## CAUTION:

- 1 eGee-Pro Refill Filter service life is for daily single usage and is limited by considerations on hygiene, damage to the mask, and increase in breathing resistance.
- 2 Ensure your hands are clean before holding filter cartridge.
- 3 Sanitize the reusable plastic protective mask cover before re-using.

## How to sanitize eGee-Pro Mask

## 1. Using soap/detergent sanitizing (OPTION 1)

Wash your mask in clean warm (around 30C) soapy water, making sure your rinse all components thoroughly before allowing them to dry naturally

## 2. Hot water sanitizing (OPTION 2)

Heat the water to boil & put the plastic reusable face mask in water for 5 ~ 10 minutes to sanitize

## 3. UV light sanitizing (OPTION 3)

Place the plastic reusable face mask with it back expose into UV light range for 15 Min, flip the mask to the front position and let it expose to UV light for another 15 minutes to sanitize.

## WARNING:

- 1 Filter cartridges are NOT for reuse. Remove the used filter and replace it with a new filter.
- 2 Not suitable for children use.

## Disclaimer

The user of this product has the sole responsibility to confirm the suitability of the product for its intended purpose and its safe use and we, our subsidiaries and/or affiliates make no representation or warranty in this regard assume no liability. To the extent that the law permits, our liability for any loss or damage that may be incurred as a result the use of or failure of the product is limited to the cost of repairing or replacing the product and does not extend to any loss or damage which may be the caused as a consequence of such use or failure. We, our subsidiaries, and/or our affiliates shall in no event be liable for economic loss or profits, indirect special or consequential damages.

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