
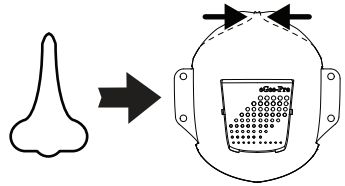
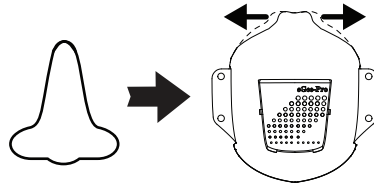
Adjust mask to fit your face contour


Pinching down on top portion of face mask to decrease the nose bridge width.

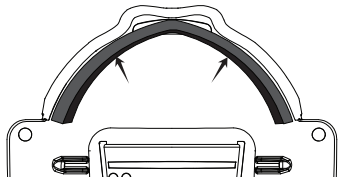Press down on top portion of face mask to increase the nose bridge width.

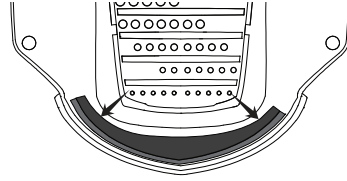
Stick the top sponge at nose piece to prevent air leak from above nose area.

(Optional) you can stick the sponge on the bottom of the mask as indicated in the picture above to prevent air leak.

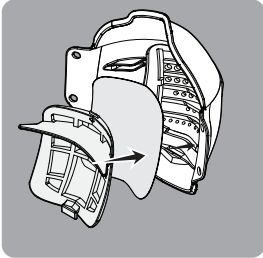How to change filter

- 1 Pull out filter compartment cap & insert filter cartridge with blue side facing outward from user's face.
- 2 Close back the filter compartment cap by inserting the bottom first and push towards until you hear a click.

CAUTION:

- 1 eGee-Pro antibacterial refill filter service life is recommended for daily single usage and is limited by considerations on hygiene, damage to the mask, and increase in breathing resistance.
- 2 Ensure your hands are clean before holding filter cartridge.
- 3 Sanitize the reusable face mask before use.

How to sanitize eGee-Pro Mask

- 1 Wash the plastic facemask with running water.
- 2 Boil the plastic facemask in water for 5 minutes to sanitize.

WARNING:

- 1 Filter cartridges are NOT for reuse. Remove the used filter and replace it with a new filter.
- 2 Not suitable for children use.

JSB TECH PTE LTD

100 Pasir Panjang Road, 100 Pasir Panjang
#06-02 Singapore 118518
Email: info@jsbtech.com

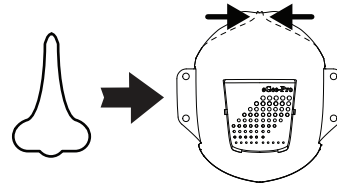
FDA approved plastic material
BPA (EC) No. 1895/2005

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